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<b>Language:</b>	English
<b>Registration MS-ANVISA:</b>	80360110001
<b>Class device:</b>	II
<b>Composition:</b>	Polymethylmetacrylate EGDMA (Ethyleneglycol dimethacrylate) Organic Biocompatible Pigments Fluorescent
<b>Appearance:</b>	Posteriors Packaging: 10 sets Anteriors Packaging: 16 sets  Moulds Upper Anteriors: B12, B13, B14, B15, B17, B21, B22, B25, B26, B32, B36, B48, B66, B2D, B3P. Moulds Lower Anteriors: B3, B4, B6, B88, B9.  Moulds Upper and Lower Posteriors: L2, L3, L4, L5 Trubyte® Shades: 60, 62, 65, 66, 67, 69, 77 e 81.
<b>Product characteristics:</b>	<ul style="list-style-type: none"> <li>*33° Angulation on posteriors;</li> <li>*Two layers (Enamel and Dentine);</li> <li>*Shade stability;</li> <li>*Manufacturing from Double Cross Linked (DCL) PMMA;</li> <li>*Absence of bubbles and porosities;</li> <li>*High mechanical; chemical and wear resistance;</li> <li>*Adherence to denture base acrylic resins;</li> <li>*Biocompatible Pigments;</li> <li>*Natural fluorescence;</li> <li>*Proportionality aesthetics, facial harmony;</li> </ul>
<b>Instructions for use:</b>	<p>Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis, as well as for individual protection.</p> <p>For an optimized bonding to the denture base resin, remove the gloss from the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.</p> <p>Recommendation: To increase the teeth durability it is highly recommended to avoid contact with alkalis, acids or other substances that can damage acrylic materials during the daily hygiene.</p>

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<b>Contraindications:</b>	There are no known side effects, if the product is used according to usual procedures.
<b>Storage:</b>	Keep away from light and heat.
<b>Indications:</b>	The Blue Lux acrylic teeth are indicated for use in total or partial dental prosthesis.
<b>Validity:</b>	6 years from manufacturing.

# Blue Dent

**Responsible Chemistry:** Rafael Gomes – CRQ 04263558  
**Registration MS-ANVISA:** 80360110001