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Language:	English
Registration MS-ANVISA:	80360110001
Class device:	II
Composition:	Polymethylmetacrylate EGDMA (Ethyleneglycol dimethacrylate) Organic Biocompatible Pigments Fluorescent
Appearance:	Posteriors Packaging: 10 sets Anteriors Packaging: 16 sets Moulds Upper and Lower Anteriors: 2D, 3D, 3M, 2N, 3N, 3P, A23, A25, A26, 263, 264, 266, 4H, 2P, 38. Moulds Upper and Lower Posteriors: 30M, 32M, 30L, 32L, 34L. Trubyte® Shades: 61, 62, 65, 66, 67 e 69.
Product characteristics:	<ul style="list-style-type: none"> *33° Angulation on posteriors; *Two layers (Enamel and Dentine); *Shade stability; *Manufacturing from High Weight Molecular PMMA; *Absence of bubbles and porosities; *High mechanical; chemical and wear resistance; *Adherence to denture base acrylic resins; *Biocompatible Pigments; *Natural fluorescence; *Proportionality aesthetics, facial harmony;
Instructions for use:	<p>Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis, as well as for individual protection.</p> <p>For an optimized bonding to the denture base resin, remove the gloss from the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.</p> <p>Recommendation: To increase the teeth durability it is highly recommended to avoid contact with alkalis, acids or other substances that can damage acrylic materials during the daily hygiene.</p>

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Contraindications:	There are no known side effects, if the product is used according to usual procedures.
Storage:	Keep away from light and heat.
Indications:	The Blue Dent acrylic teeth are indicated for use in total or partial dental prosthesis.
Validity:	6 years from manufacturing.

Blue Dent

Responsible Chemistry: Rafael Gomes – CRQ 04263558
Registration MS-ANVISA: 80360110001