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Language:	English
Registration MS-ANVISA:	80360110001
Class device:	II
Composition:	Polymethylmetacrylate EGDMA (Ethyleneglycol dimethacrylate) Organic Biocompatible Pigments Fluorescent
Appearance:	Posteriors Packaging: 05 sets Anteriors Packaging: 08 sets Moulds Upper Anteriors: R1, R2, R3, R4, R5, V3, V4, A2, A3, Z1, Z3, Z4, C1, C2, X1, X2, X4. Moulds Lower Anteriors: T2, T3, T5, T6, T7, T9. Moulds Upper and Lower Posteriors: S2, S3, S4, S5. Shades: 1A, 2A, 3A, 2B, 4B, 1C, 2C, 1D, 1E and 3E.
Product characteristics:	<ul style="list-style-type: none"> *33° Angulation on posteriors; *Three layers (Enamel, Dentine and Cervical); *Shade stability; *Manufacturing from High Weight Molecular PMMA, combined with Double Cross Linked (DCL); *Absence of bubbles and porosities; *High mechanical; chemical and wear resistance; *Adherence to denture base acrylic resins; *Biocompatible Pigments; *Natural fluorescence; *Proportionality aesthetics, facial harmony;
Instructions for use:	<p>Follow the usual procedures for choice, waxing-up, articulation, polymerization, infection-control and maintenance of the prosthesis, as well as for individual protection.</p> <p>For an optimized bonding to the denture base resin, remove the gloss from the tooth base with a grinding instrument. After insulating the flask and prior to pack the acrylic dough, clean the base surface of the tooth using methylmethacrylate monomer.</p> <p>Recommendation: To increase the teeth durability it is highly recommended to avoid contact with alkalis, acids or other substances that can damage acrylic materials during the daily hygiene.</p>

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Contraindications:	There are no known side effects, if the product is used according to usual procedures.
Storage:	Keep away from light and heat.
Indications:	The TRISUPER acrylic teeth are indicated for use in total or partial dental prosthesis.
Validity:	6 years from manufacturing.

Blue Dent

Responsible Chemistry: Rafael Gomes – CRQ 04263558
Registration MS-ANVISA: 80360110001